


ORAcollect saliva sampling and postal instructions


Follow steps 0-7 for sample collection and reply paid postage




No food 15-30 minutes prior

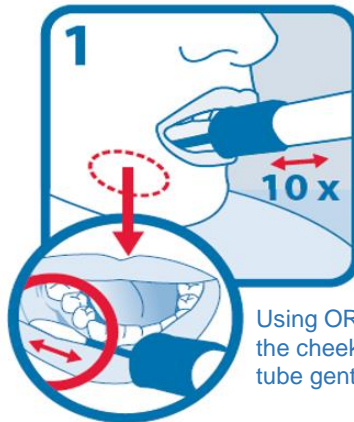

Using ORAcollect tube, sample where the saliva pools in the cheek; 10 x on the left and 10 x on the right, moving tube gently backwards and forwards

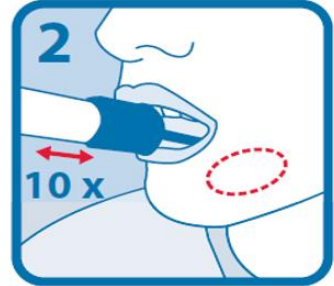


Unscrew lid

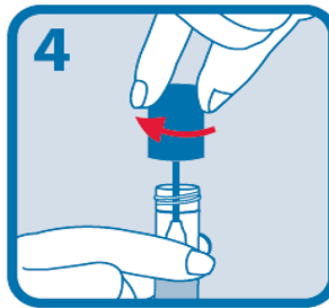


Invert and screw lid tight



Invert 15 times

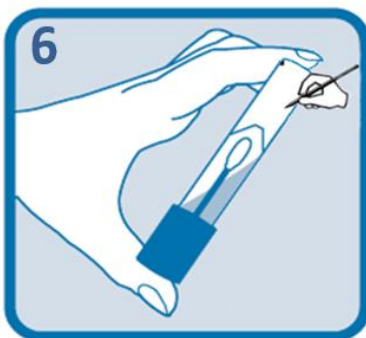

Label the tube with name, date of birth and date collected

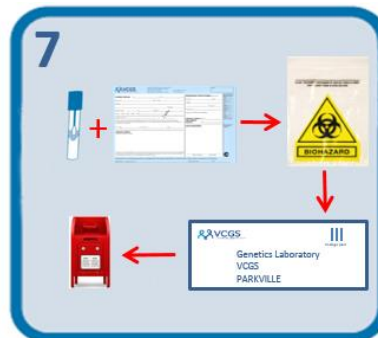

Place tube and request slip into supplied bag (which contains an absorbent pad). Place this into supplied envelope and post in regular mail.



<http://www.dnagenotek.com/ROW/support/ciOCR100.html>